Rolko-dribo
Trinkflasche mit Halter
Drink Bottle with Holder
Drikkedunk med holder
Bidon met Houder
Gourde et support bidon

	Gebrauchsanweisung	2
	Instruction manual	4
	Gebruiksaanwijzing	6
	Brugsanvisning	8
	Mode d'emploi	10

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Dear Customer,

we thank you for your confidence in our company by purchasing one of our products. Rolko-dribo - our drink bottle with holder for rollators and wheelchairs was developed according to the highest standards, using the most advanced technologies and with the thoroughness and the team spirit of our employees. Please take note of instructions listed below and consider specially the safety instructions to guarantee the proper use of our product.

01. Purpose and Indication

Our drink bottle with holder was specially designed to make the everyday life of wheelchair and walker users more enjoyable. The universal clamp of the holder is suitable for common wheelchairs and rollators with a round tube diameter of 16 up to 32 mm. The holder is 360 ° rotatable and allows so many fixation possibilities.

02. Contraindication

The drink bottle and the supplied holder are unsuitable for all applications which deviate from the intended use and may lead to dangers.

03. Safety Instructions

- These operating instructions must be read before using the product.
- Check the drinking bottle and the holder for transport damages before the first use.
- For safety reasons, do not use damaged bottles or holders.
- Before the first use, rinse the water bottle 1 - 3 times with hot water or in the dishwasher (at least with 65 °C).
- Caution: Hot liquids can cause scalding. Not suitable for microwaves.
- Before using the bottle, make sure that the lid is tightly screwed and thus securely closed.
- Check the tightness before use by turning the bottle upside down.
- Fill the bottle to max. 2.5 cm below the upper edge.
- Fermenting, alcoholic or nutrient-rich drinks can build up excess pressure in the bottle. Let this pressure escape by gently and slowly turning the lid.
- Make sure that the wheelchair / walker is fixed during assembly and cannot move unexpectedly.
- After mounting, make sure the holder is firmly and securely mounted to the wheelchair / rollator tube.
- The set is not suitable for children under 3 years.
- The effects of UV radiation, mechanical stress, natural aging or careless handling can cause premature wear of the product to which the manufacturer has no direct influence.

04. Attachment

First, make sure your wheelchair or rollator is locked against an unwanted rolling. Select a suitable location on the wheelchair / rollator to attach the holder. When choosing the position, make sure that you can easily reach the drinking bottle.

Loosen the universal clamp by folding the lever upwards (see fig. 1) and the turning the lever counter clockwise to increase the clamp opening (see fig. 2) until the clamp fully encloses the tube of the wheelchair / rollator frame.

ATTENTION: open the clamp by turning the lever only as far as necessary! If you have opened the clamp too far, the screw of the lever will release from the opposite side. In the lower half of the clamp there is a steel pin with internal thread. If this has dropped out, put it back in the slot provided. Turn it so that the thread fits the screw of the lever. Now the lever can be screwed in again.

Slide the holder to the desired position and tighten the clamp by turning the lever clockwise (fig.2). Always turn the lever so

far that it points over the clamp! Now fold it down (fig.3). If it is too hard to fold the lever down, release the lever one turn counter clockwise and fold it down again.

Depending on where you want to attach the holder to the wheelchair / walker, you can turn the clamp in 90° steps. To do this, remove the bottle and clamp from the holder by completely loosening the hexagon socket screw M3 (fig. 4) on the inner side of the holder. Then turn the clamp to the desired position and screw it onto the holder again (fig. 4). The holder should be positioned so that the bottle is in an upright position inside the holder. Please note that no Allen key is included in the delivery!

Fig. 1

Fig. 2

Fig. 3

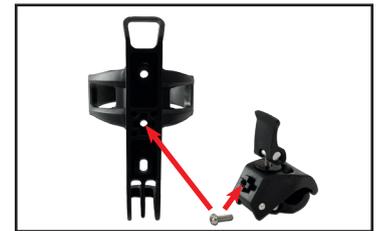
Fig. 4

05. Cleaning

You may clean the bottle and holder from the outside with a damp cloth. The bottle is dishwasher safe up to 65 °C. If the bottle is filled with juice, sports drink or other liquids than water, clean the bottle with a mild detergent after each use. After cleaning, let the bottle as well as the holder dry sufficiently.

06. Maintenance

Check the bottle and the holder regularly and at least once in a month for visual defects. Make sure that the set is undamaged and all materials are in perfect condition. In case of signs of wear out or defect, the bottle and holder must be disposed of immediately!

07. Technical Data

Item no.: U-FL-H750

Scope of delivery: set with drinking bottle and holder incl. universal clamp.

Bottle dimensions (height x width): 230 x 74 mm, capacity: 750 ml, weight: 80 g, weight incl. holder: 252 g.

Materials used (bottle): body LDPE, lid PP, BPA-free and without plasticisers.

Materials used (bracket): plastic, galvanized steel and aluminium.

Disposal: the drinking bottle as well as the holder must be disposed of in accordance with the laws and regulations of your country in a proper and professional manner.

Instruction manual: Rolko-dribo U-FL-H750 / Revision 2022-05-23 / Rev. 03
© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Germany, Germany
Reprint, also in extracts, only with prior permission by the editor. All rights reserved.
Alterations for reasons of technical development reserved. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances. Errors excepted.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Germany, Allemagne