## Handgriff GR-ERGO-GS

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## Handgrips GR-ERGO-GS

Dear Customer,

we thank you for your confidence in our company by purchasing one of our products. We are convinced that our handgrips entirely meet the requirements of the daily nursing routines. Our handgrips were developed according to the highest medical standards using the most advanced technologies. We are confident that our handgrip makes everyday life of nursing staff, accompanying persons and people in need of care easier and more pleasant. Please take note of instructions listed below and consider specially the safety instructions to guarantee the proper use of our product.

### 01. Purpose and Indication

The ERGO handgrips for wheelchairs provide more comfort for the hands of accompanying persons of wheelchair users. In addition, the handgrips have an ergonomic shape that is optimised for the inside hand so that the ulnar nerve is relieved.

### 02. Contraindication

Our handgrips are not suitable for all applications that deviate from the intended use and/or may lead to danger.

### 03. Safety Instructions

- Instruction manual: please mind the instructions listed in this manual before using this product.
- Defective or damaged handgrips must not be used and need to be replaced immediately.
- Please note that the handgrips may get loose from the wheelchair push handles when the ambient temperature or humidity is very high. Always make sure that the handgrips are firmly attached to the push handles!
- Risk of injury: If you are pulling a person sitting in a wheelchair over an obstacle (for example, a kerb/curb), do not hold on to the handgrip but always on a part of the wheelchair that is fixed to the wheelchair frame!

### 04. Maintenance Interval

Check on a regular basis the integrity of the handgrips and make sure that they always are firmly attached to the push handles. Defective or damaged handgrips may not be used and must be replaced immediately.

### 05. Assembly

Before assembling please read this instruction manual carefully and check the product for any damages that may have occurred during the transport.

The handgrips GR-ERGO-GS are suitable for push handle tubes with a diameter of 22 mm.

First remove the possibly existing handgrips of the wheelchair. Clean the tube ends under the sliced hand grips. Use only clear water and no detergents / cleaning agents. After that dry the tube ends with a clean cloth.

For the assembly, slide the handgrips onto both push handle tubes of the wheelchair. Make sure that the light grey soft surface of the handgrips points into the thumb balls direction of the wheelchair accompanying person.

Please note that handgrips must be firmly attached to the push handles so that they can not slide off. For this reason, the mounting may be a little difficult. We therefore recommend using our mounting aid spray "Fix4Grip": Simply spray into the inner part of the handgrips and then insert them into the push handle tubes. After approx. 2 hours, the handgrips are securely fixed to the wheelchair's push handles.

The proper fixation of the handgrips onto the push handles need to be ensured after the assembly and also before each user of the wheelchair.

## 06. Technical Data

Item no.: GR-ERGO-GS

Scope of delivery: 1 pair

Length: 120 mm

Inner diameter: 22.2 mm

Own weight (pair): 0,164 kg

Colour: black / lightgrey

Form: ergonomic

Applied materials: TPE / Kraton rubber (free from phthalate, PAH, latex und caoutchouc)

Disposal: the handgrips need be properly disposed, according to the laws and regulations in your country.

To download this and further instruction manuals of our products please visit: [instructions.rolko.com](http://instructions.rolko.com).

**Operating Instructions: Handgrip GR-ERGO-GS / Revision 2017-10-19**

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved.

Alterations for reasons of technical development reserved. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances. Errors excepted.

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**© Rolko Kohlgrüber GmbH, Industriestr. 14-16, 33829 Borgholzhausen, Duitsland**