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Abduction Wedges and Holders

01. Purpose, application

The abduction wedges and the corresponding holders from Rolko are accessories for wheelchairs. With an abduction wedge and its holder, the positioning of the legs when sitting in the wheelchair is optimised and the hip joints are relieved.

02. Contraindication

The evaluation of the necessity, possibilities of assembly, selection of the correct model of the abduction wedge holder and optional abduction wedge, assembly and adjustment of the correct position must be carried out by trained specialists.

03. Safety instructions




- These instructions must be read and followed before using the product.
- Check the abduction wedge and or the holder for damage before using it for the first time.
- A gap between the inserted abduction wedge and the seat is not permitted. There is a risk of body parts being trapped.
- A damaged / defective abduction wedge and or holder must never be used, but must be replaced immediately.
- The instructions are part of the abduction wedge and or holder and must be passed on when they are passed on.
- Serious incidents in connection with this product occur, those must be reported to Rolko Kohlgrüber GmbH and the competent authority of the EU member state in which the product user lives.

04. Use


Attach the support tube with the fastening plate under the seat plate of the wheelchair frame or the seat shell. The screws required for this are not included in the scope of delivery. The push-in sleeve with quick release receives the bracket for the abduction wedge. The edge-fixing hanger is clamped at the desired height using the quick release.

The depth of the insert sleeve can be adjusted by loosening the wing screw on the support tube and by tightening the wing screw can be fixed in the desired position. The optional abduction wedge is fastened in the desired position with the help of the two enclosed universal screws. A space of at least 25 mm from the edge of the abduction wedge must be secured (Fig. 1).

The abduction wedge holder is available in 2 different versions. These versions differ only in the support bracket for the abduction wedge. The support bracket, on which the abduction wedge is attached to, is 80 mm long in the HA-AK-1 model (Fig. 2) and 160 mm long in the HA-AK-2 model (Fig. 3).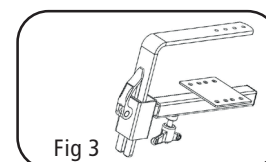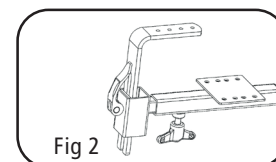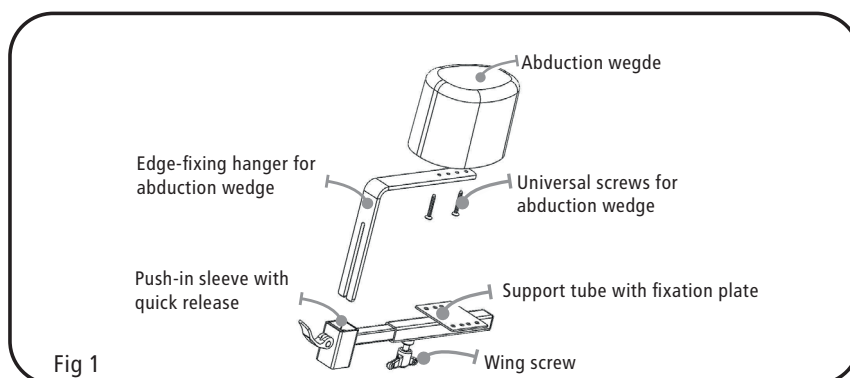

By loosening the quick release, the support bracket with abduction wedge can be removed quickly. This makes getting in and out of the wheelchair easier. By inserting the support bracket into the insert sleeve, the support bracket is clamped in the desired position by locking the quick release. It should be noted that no gap between the seat and the abduction wedge should exist where parts of the body can get caught.

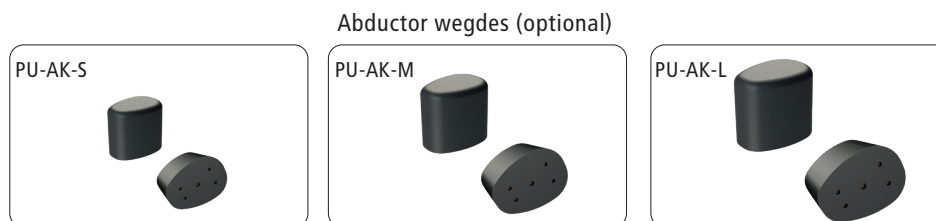05. Cleaning and desinfection

For cleaning, the abduction wedge holders can be wiped with a damp cloth, if necessary with the addition of a mild detergent. Use lukewarm water and, if necessary, non-abrasive, pH-value neutral, commercially available household cleaners. If necessary, the abduction wedge holders can be cleaned with commercially available disinfectants by spray or wipe disinfection. The use of cleaning agents and disinfectants with chlorine or organic or inorganic substances with active chlorine is not permitted. The following cleaning agents and disinfectants or processes are not permitted for the abduction wedges (made of polyurethane):

- hot water, steam,
- concentrated acids and alkalis,
- alcohol,
- chlorinated hydrocarbons,
- ammonia.

06. Maintenance

Check the function of the abduction wedges and the associated holding system before each use and make sure that they are undamaged and in perfect condition. Before each use, check:

- the tight fit of the fastenings,
- the correct setting of the abduction wedge holder,
- the product visually for damage.

07. Reuse




The product is suitable for reuse. Clean the abduction wedge and or holder system according to the specifications under "05. Cleaning and disinfection" and carry out the maintenance according to section "06. Maintenance".

08. Disposal

All metal parts of the abduction wedge holder are accepted by the recyclable materials trade. The polyurethane abduction wedges can be safely disposed of with household waste. If possible, dismantle the abduction wedge holder into its individual parts to enable the materials to be sorted.

09. Technical data

Item no.:	abduction wedge holders: HA-AK-1, HA-AK-2 abduction wedges (optional): PU-AK-S, PU-AK-M, PU-AK-L
Scope of delivery:	abduction wedge holder, manual
Temperature range:	-20°C up to +50°C, relative humidity is max. 80%
Used materials:	steel, polyurethane, wood
Label:	

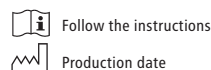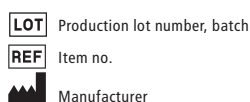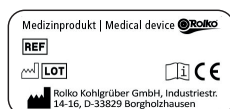As an accessory for class 1 medical products, the product has a CE mark in accordance with Regulation (EU) 2017/745 (MDR) and thus complies with all of the German and European safety requirements applicable to the product. The product complies with the relevant sections of the DIN EN 12182 standard. The EC declaration of conformity is available for you to download at instructions.rolko.com.

Instruction manual: Abduction wedge holders HA-AK-1, HA-AK-2, abduction wedges: PU-AK-S, PU-AK-M, PU-AK-L.

Status 2022-02-23 / Rev. 02 / Basis-UDI-DI: 426067605PUAKH0000000008E

© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Germany. Reprinting, even in part, only with the prior approval of the publisher. All rights reserved. We reserve the right to make changes due to further technical developments. All technical data are nominal data and are subject to design and manufacturing tolerances.

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