Tetra-Griff für Quick-Release-Achsen
Tetra handle for quick release axles
Tetra-handgreep voor quick-release assen
Tetra-håndtag til quick-release-aksler
Poignée Tetra pour axes Quick Release
Maniglia Tetra per assi a sgancio rapido
Asa Tetra para ejes de liberación rápida

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CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen

01. Intended purpose, area of application

The Tetra handle for quick release axles (QRA) by Rolko is an accessory for wheelchairs. It is used in combination with a quick release axle. If the user has restricted use of their hand, the Tetra handle makes it easier to unlock the QRA to remove the wheel of the wheelchair.

02. Contraindication

The Tetra handle can be used only for the common QRA with a M12x1 thread and a knob spring deflection of approx. 4.5 mm. It may not be used for all other axles. Professional technicians must evaluate whether the device is necessary, whether it cannot be installed, and whether the position is set correctly.

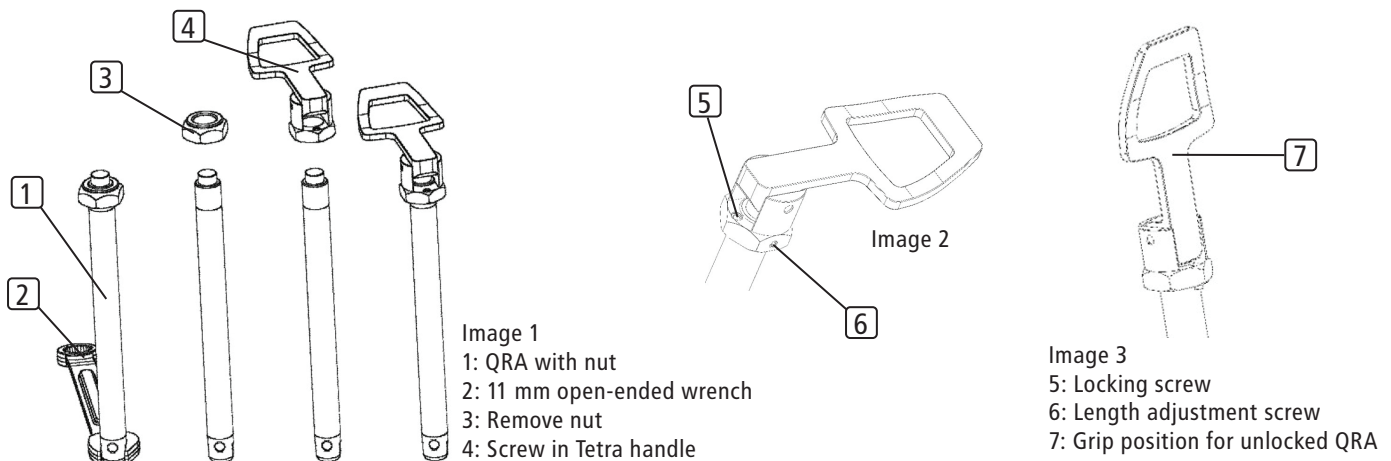
03. Safety information

- These instructions for use must be read and observed before using the product.
- The instructions for use are considered part of the product, and must be provided with the product if it is passed on to another user.
- The Tetra handle may increase the width of the wheelchair. There is a risk of collisions.
- Ensure that the lever is in the locked basic position after the wheel is installed. If the axle is unlocked, the wheels can come loose. Because the Tetra handle protrudes far beyond the wheel, there is a risk of collision which could result in damage to the Tetra handle, the wheelchair, or other objects.
- Serious incidents involving this product must be reported to Rolko Kohlgrüber GmbH as well as the responsible authority of the EU member nation where the user is located.

04. Assembly and handling

Assembly

Before using the product for the first time, please inspect it for any damages.

First, remove the nut on the QRA and exchange it for the Tetra handle (see image 1). The nut (3) of the QRA is unscrewed from the QRA (1) using an 11 mm open-ended spanner (2) and a 19 mm ring spanner or open-ended spanner. The Tetra handle (4) is screwed onto the QRA in place of the nut until the Tetra handle lightly touches the knob in the position shown in image 1. After the Tetra handle has been screwed into position, the position is fixed by tightening the locking screw (5) slightly. A 1.5 mm Allen key is used for this purpose. Then the wheel of the wheelchair is inserted into the wheelchair with the axle and the clearance is checked in the axial direction. A small amount of clearance is permitted. The axial clearance can be reduced by screwing in the adjustment screw (5). A 2 mm Allen key is used for this purpose. The adjustment screw may protrude a maximum of 1.5 mm on the bottom. Using a precision mechanic oil free from resins and acids on all movable parts of the Tetra handle and axle is permitted to support the function.

Handling

Image 2 shows the Tetra handle in the basic position. The axle is locked and the wheel of the wheelchair is secured in the inserted position. The axle must be unlocked to remove and insert the wheel. The unlocked position of the Tetra handle is shown in image 3. In this position, the axle can be removed or inserted with the wheel of the wheelchair. The Tetra handle is shaped so that a finger or thumb will fit into the opening to make it easier to remove the axle if the user has restricted use of their hand.

Please note the following:

- The axles must be locked while the wheelchair is in use (position corresponding to image 2). If the axle is unlocked, the wheels can come loose. Handles that stick out result in an increased risk of collision, which could damage the wheelchair and other objects. Personal injuries are possible due to an accident.
- The Tetra handle can extend slightly beyond the hand rims slightly even in locked position. Note the additional width to avoid damage to the handle or other objects.
- The Tetra handle can be used only for the common QRA with a M12x1 thread and a knob spring deflection of approx. 4.5 mm.
- Professional technicians must handle the installation and adjustments.

05. Cleaning

The Tetra handle and axle are coated with a precision mechanic oil free from resins and acids for cleaning; excess oil should be wiped off with a cloth. The aluminium operating lever can then be cleaned of oil using a cloth and a degreasing cleaner.

06. Maintenance

The product does not have a limited service life; therefore, it is suitable for reuse after being subjected to regular and thorough inspections. Defective or damaged Tetra handles should never be used, and must be replaced immediately. Complete a functional test once each quarter. Clean the lever and axle according to section "05. Cleaning".

07. Reuse

The product is suitable for reuse. The Tetra handle may be damaged due to wear or careless handling. Clean the axle according to section "05. Cleaning" and dry the axle in a dry area protected from dust.

Before reusing the product, carry out maintenance as described in section "06. Maintenance".

08. Disposal

Disassemble the Tetra handle as far as possible, and recycle the individual components.

09. Technical data

Material: Aluminium, steel, stainless steel
Dimensions: 78 x 39 mm (length x width)
Thread: M12x1

Label

	Medical device
	Production date
	Item number
	Manufacturer
	Unique product identifier

As an accessory for class 1 medical devices, the product bears the CE mark according to Regulation (EU) 2017/745 (MDR) and therefore conforms to all German and European safety requirements applicable for the product. The product fulfils the relevant sections of standard DIN EN ISO 21856. These instructions for use and the EC Declaration of Conformity for the product are available in pdf format on www.rolko.com. You can also use the read aloud function of your browser.

Instructions for use: Tetra handle for quick release axles / QRA-TETRA-01 / 2024.03.06 / Rev. 1
Basic UDI-DI 426067605RSZ0000000000PB

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Full or partial reprints are permitted only by prior approval of the author. All rights reserved. The delivered product may deviate from the images shown in these instructions. All technical data are nominal data, and are subject to design- and manufacturing-related tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen