## RG-ROL-01 Rollator-Rückengurt

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## RG-ROL-01 Rollator-Backstrap

### 01. Purpose and Indication

The rollator back strap is designed for use on the rollator. It enables the back to be supported when sitting on the rollator. The cast iron diameter for attachment must be 22 mm.

### 02. Contraindication

Our rollator back strap is unsuitable for all applications that deviate from the intended use and can lead to dangers. The rollator back strap can only be used on rollators with a seat and with a round tube for attachment with a diameter of 22 mm. It is unsuitable for all other rollators.

### 03. Safety Instructions


- This instructions for use must be read and observed before use.
- Only sit on the rollator with the brakes on!
- The installation must be carried out by trained personnel.
- The instructions for use are part of the rollator back strap. If this is passed on, this must also be given.
- If serious incidents occur in connection with this product, the incidents must be reported both to Rolko Kohlgrüber GmbH and to the competent authority of the EU Member State in which the user or patient is based.

### 04. Operation


Read the instructions carefully before use. Please check the item for damage and for completeness. Proper seating must be ensured after assembly.


Apply the rollator parking brake and remove its push handle tubes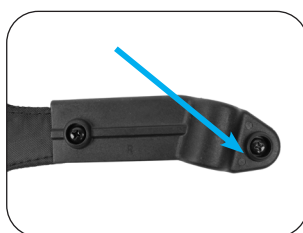

Release the Screw the pipe clamps of the backstrap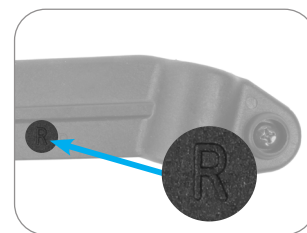

Pay attention to the correct side marks (L: left / R: right) on the clamps

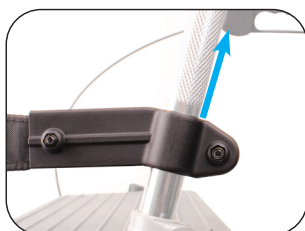Carefully push the clamps from bottom to top through the tubes


Fix the push handle tubes back at the correct height on the Rollator.


Once the desired belt height is reached, tighten the Screw the pipe clamps

## Adjustment of the Belt Length

The belt is 575 mm long on delivery. The adjustment of the correct belt length must be carried out by trained personnel. To adjust another length, proceed as follows: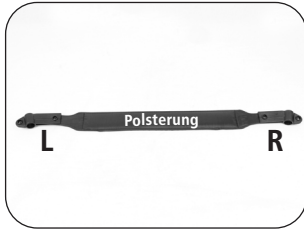


Lay the strap with the padded surface upwards

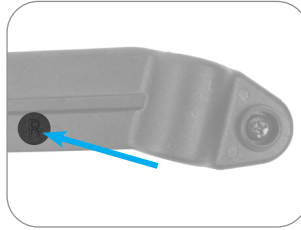
Pay attention to the side marks (L: left / R: right)

Loose the screws to which the strap is attached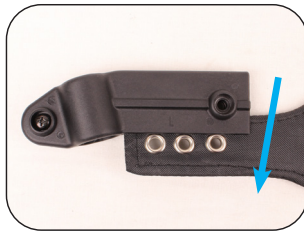

Pull the strap down out of the clamps

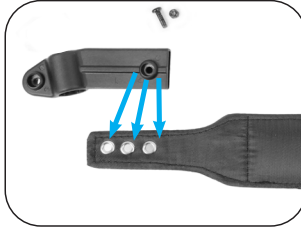Select the appropriate hole depending on the desired length.

Is the max. belt length adjusted, the screws must be tightened so that the belt cannot tip down.

## 05. Care, Cleaning and Disinfection

For cleaning, the belt can be wiped with a damp cloth, if necessary with the addition of a mild detergent. Use lukewarm water and, if necessary, non-abrasive, pH-neutralized, commercially available household cleaners. If necessary, the belt can be disinfected with household disinfectants by spray or wipe disinfection. Be sure to follow the manufacturer's instructions for use. Do not use detergents and disinfectants that contain concentrated acids and bases, chlorinated hydrocarbons and chlorine.

## 06. Maintenance and Servicing

Check the seams and the strength of the screw connection for integrity every month. Defective or damaged belts must never be used and must be replaced immediately.

## 07. Re-use

The product is suitable for reuse. The effects of weather and UV radiation, mechanical stress as well as natural aging or careless handling can cause premature wear, over which the manufacturer has no direct influence. Please note item 06 Maintenance. Clean the belt before each reuse (see: Care, cleaning and disinfection).

## 08. Disposal

The belt can be safely disposed of with household waste.

## 09. Technical Data

item no.: RG-ROL-01  
 Length: 575-640 mm  
 Belt width: 65 mm  
 pipe diameter: 22 mm  
 Dead weight: 0.2 kg

identification

 Production batch

 Item no.

 Manufacturer

 Consider operating instructions

Permitted environmental conditions:  
 Temperature in the range of -10 °C bis +40 °C  
 Relative humidity is max. 80%

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN 12182. The EU declaration of conformity is available for download at [instructions.rolko.com](http://instructions.rolko.com).

**Operating Instructions: RG-ROL-01 (Rollator-Backstrap) / 2021-05-28 / Rev. 06**  
**Basis-UDI-DI: 426067605RG-ROL000000003Q**

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Germany