**Hüftgurt**  
**Hip Strap**  
**Heupgordel**  
**Hofterem**  
**Sangle de hanche**  
**Cintura addominale**

**Rolko**  
**EasyTRANSFER**

**CE**      **TFG-L**

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Dear customer,

The Rolko team thanks you for the trust you have shown in us by purchasing our product. In purchasing the **Rolko-EasyTransfer hip strap**, you have chosen a highly functional product that makes everyday life easier for care users, their carers and companions. Our hip strap has been developed in line with the latest standards, using state-of-the-art technologies, and with the thoroughness and team spirit of our staff. We are certain that its high use value and robust quality will provide the best possible support for care users and those who assist them. Below, care users are referred to as "users", while their assistants and carers are referred to as "helpers".

## 01. Intended purpose, area of use

The hip strap is used for walking training and as a transfer aid for people with significant or full impairment when changing or transferring the body position. Users who suffer from immobility, significant damage to their body perception or neurological disorders can be transferred in a controlled way with the support of a helper and using the hip strap. With its many handle options, the hip strap ensures that the user and helper have a secure hold when standing up, sitting down or lying down, during transfer between bed and wheelchair, and during walking training in both domestic and care settings.

## 02. Contraindication

The hip strap is not suitable for any applications that deviate from proper use and can lead to hazards. The maximum user weight is 130 kg. The hip strap must not be used without trained helpers.

## 03. Safety information

- The helper must read and adhere to this instruction manual before using the product!
- Check the hip strap before using it for the first time and after every wash!
- Never use a damaged/defective hip strap – replace it immediately.
- The user must be supported and held sufficiently in order to prevent slips and falls.
- Ensure that the plastic buckle is located on the stomach! There should be as little pressure as possible on the buckle during use.
- As a helper, ask the user to support the transfer where they can.
- If the hip strap is to be used to support walking training, it is vital to first ensure that the user can stand up straight and support their weight.
- Soiling and wetness should be avoided!
- Do not leave the hip strap on the floor (trip hazard!).
- The instruction manual is part of the hip strap and must be passed on together with the hip strap.
- Serious incidents in connection with this product must be reported to Rolko Kohlgrüber GmbH and the authority responsible in the EU member state in which the user is based.

## 04. Use

The example below explains transfer from bed to wheelchair. Using the hip strap may also be useful for other changes of position and to support walking training. Also available: For the best possible person transfer, we recommend using the leg strap in combination with our hip strap (TFG-L), our slide boards (RBK-700-01, RBK-KV, RBK-KV-2) and our turntable (DRT-400).

Positioning the strap:

- Place the hip strap around the hips or waist of the user and close the buckle in front of the stomach (Figure 1). Take hold of the end of the strap and pull the hip strap tighter, until it is lying tightly yet comfortably around the user. You may need to tighten the strap again once the user has stood up (when using for walking training).
- The helper can wear a second hip strap for the user to hold on to. Warning: The user can be a hazard; if they lose their balance, they can cause the helper to fall, resulting in injury.

Use the docking station for the strap end:

- To close: Place the strap end close to the docking station so that it locks into position by itself.
- To release: Move the strap end away from the lock. The strap can only be released by pushing it to the side, not by pulling from the front!

Removing the strap:

- Loosen the strap and open the buckle.

Shifting positions:

- Push the wheelchair as close to the bed as possible in order to reduce the distance to be bridged to a minimum. Then secure the wheelchair so that it cannot roll away.
- To assist with standing from sitting, the helper(s) should position themselves to the side or in front of the user and take hold of the loops on the user's hip strap. If possible, the user should lean forward and assist the process themselves. The helper – or the helpers on both sides – can carefully pull the user up from sitting to standing (Figure 2).
- The user can then be turned towards the wheelchair. Once the required position in front of the wheelchair has been reached, the user can slowly be seated in the wheelchair (Figure 3).

Walking training:

- If the user is standing, the helper(s) should position themselves to the side. The helper takes hold of a handle in on the hip strap a suitable place on the back with one hand and holds the user's hand with the other (Figure 4).

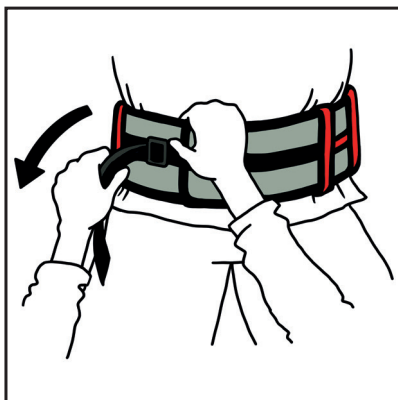
Figure 1

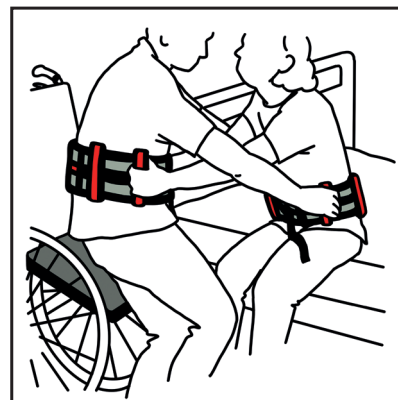
Figure 2

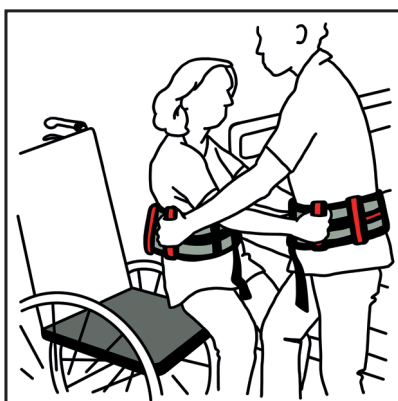
Figure 3

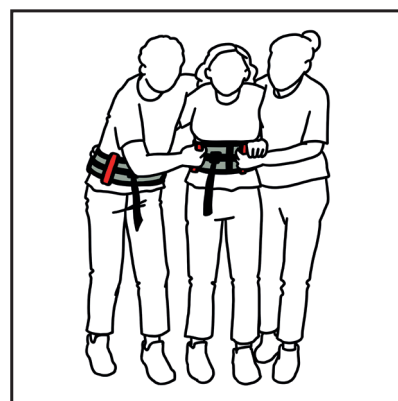
Figure 4

## 05. Cleaning and disinfection

Remove small areas of soiling with a damp cloth and non-abrasive, pH-neutral, household cleaner. The hip strap can be washed in a washing machine at max. 40°C, max. 800 rpm, without fabric softener. If required, the hip strap can be cleaned with household disinfectant by spraying or wiping. Always follow the manufacturer's instructions. If the hip strap is damp, allow it to dry before next use. Chlorine or organic or inorganic substances with active chlorine must never be used, as these can change the material properties of the product and thus cause potential material failure and therefore hazards.

## 06. Maintenance

Check the hip strap carefully for visible defects before each use. Ensure that the fabric and the seams of the hip strap, especially the loops, are undamaged and in a flawless state. Never use a damaged or defective hip strap – replace it immediately.

## 07. Reuse

The product is suitable for reuse. Clean and disinfect the hip strap before each use and conduct maintenance as per Section 06.

## 08. Disposal

The hip strap can be disposed of in household waste.

## 09. Technical data

Item no.: TFG-L

Scope of delivery: hip strap, instruction manual


Dimensions: length: 1,200 mm, width 130 mm

Maximum user weight: 130 kg

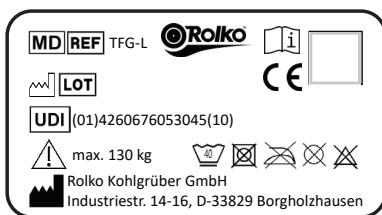Intrinsic weight: 250 g

Washable: at 40°C and max. 800 rpm

Materials used: polypropylene, polyester, polyurethane, polyoxymethylene, polyamide

-  Medical product
-  Item number
-  Production lot number, batch
-  Unique device identification
-  Manufacturer
-  Follow instruction manual

-  Wash at 40°C
-  Do not tumble dry
-  Do not iron
-  Do not dry clean
-  Do not bleach

As a Class 1 medical product, the product carries the CE label in accordance with Regulation (EU) 2017/745 (MDR) and thus meets all German and European safety requirements applicable to the product. The product meets the relevant sections of the DIN EN ISO 21856 standard. The EC Declaration of Conformity is available for download at [instructions.rolko.com](http://instructions.rolko.com).

**Instruction manual: Rolko-EasyTransfer ship strap - TFG-L / dated 2024-03-28 / Rev. 01**

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Basic UDI DI: 426067605TGEASYTRANSFER38

Reproduction, in whole or in part, only with prior permission of the publisher. All rights reserved. Subject to change due to technical development. All technical data is nominal data and subject to tolerances due to construction and manufacture.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen
Importatore CH: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen