Beingurt  
Leg Strap  
Beengordel  
Benrem  
Sangle de jambe  
Cinturino per gambe

**Rolko**  
**EasyTRANSFER**

CE

TFH-L

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Dear customer,

The Rolko team thanks you for the trust you have shown in us by purchasing our product. In purchasing the **Rolko-EasyTransfer leg strap**, you have chosen a highly functional product that makes everyday life easier for care users, their carers and companions. Our leg strap has been developed in line with the latest standards, using state-of-the-art technologies, and with the thoroughness and team spirit of our staff. We are certain that its high use value and robust quality will provide the best possible support for care users and those who assist them.

Below, care users are referred to as "users", while their assistants and carers are referred to as "helpers".

## 01. Intended purpose, area of use

The leg strap is used as a transfer aid for people with significant or full impairment when changing or transferring the body position. Users who suffer from immobility, significant damage to their body perception or neurological disorders can be transferred in a controlled way in both domestic and care settings with the support of a helper and using the leg strap (as well as other aids such as the hip strap, turntable or slide board, if required).

## 02. Contraindication

The leg strap is not suitable for any applications that deviate from proper use and can lead to hazards. The maximum user weight is 130 kg. The leg strap must not be used without trained helpers.

## 03. Safety information

- The helper must read and adhere to this instruction manual before using the product!
- Check the leg strap before using it for the first time and after every wash!
- Never use a damaged/defective leg strap – replace it immediately.
- The user must be supported sufficiently in order to prevent slips and falls.
- As a helper, ask the user to support the transfer where they can.
- Soiling and wetness should be avoided.
- Do not leave the leg strap on the floor (trip hazard!).
- The instruction manual is part of the leg strap and must be passed on together with the leg strap.
- Serious incidents in connection with this product must be reported to Rolko Kohlgrüber GmbH and the authority responsible in the EU member state in which the user is based.

## 04. Use

The example below explains how to transfer a user from bed to wheelchair using our turntable and hip strap. Also available: For the best possible person transfer, we recommend using the leg strap in combination with our hip strap (TFG-L), our slide boards (RBK-700-01, RBK-KV, RBK-KV-2) and our turntable (DRT-400).

The use of the leg strap is also useful for other changes of position, such as to the toilet. Another option for using the leg strap is when the legs of a recumbent patient or user need to be moved in the bed (Figure 1).

- Push the wheelchair as close to the bed as possible in order to reduce the distance to be bridged to a minimum. Secure the wheelchair so that it cannot roll away.
- The user sits on the edge of the bed with their feet on the floor. Place the feet a foot's width apart on the floor or turntable (if using).
- Position the leg strap around the patient's legs at calf height and secure with the hook-and-loop fastener.
- The helper places their foot between the feet of the user and supports the leg strap with their shin (Figure 2). The helper can then carefully pull the patient from seated up into a standing position. The leg strap creates a lever effect that makes it much easier to lift the patient.
- The user needs to carry their own weight and stand firmly – with or without support.
- The user can now be turned towards the wheelchair (Figure 3). Once the required position in front of the wheelchair has been reached, the user can slowly be seated in the wheelchair.

Figure 1

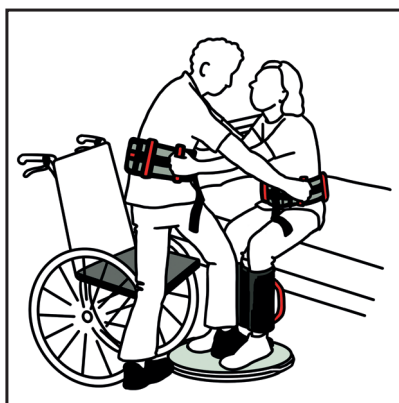
Figure 2

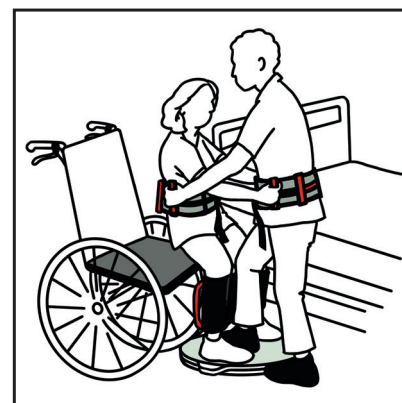
Figure 3

## 05. Cleaning and disinfection

Remove small areas of soiling with a damp cloth and non-abrasive, pH-neutral, household cleaner. The leg strap can be washed in a washing machine at max. 40°C, max. 800 rpm, without fabric softener and with the hook-and-loop fastener closed.

If required, the leg strap can be cleaned with household disinfectant by spraying or wiping. Always follow the manufacturer's instructions. If the leg strap is damp, allow it to dry before next use.

Chlorine or organic or inorganic substances with active chlorine must never be used, as these can change the material properties of the product and thus cause potential material failure and therefore hazards.

## 06. Maintenance

Check the leg strap carefully for visible defects before each use. Ensure that the fabric and the seams of the leg strap, especially the loops, are undamaged and in a flawless state. The hook-and-loop fastener must close securely.

Never use a damaged or defective leg strap – replace it immediately.

## 07. Reuse

The product is suitable for reuse. Clean and disinfect the strap before each use and conduct maintenance as per Section 06.

## 08. Disposal

The leg strap can be disposed of in household waste.

## 09. Technical data

Item no.: TFH-L

Scope of delivery: leg strap, instructions for use


Dimensions: Length 1,000 mm, width 220 mm

Maximum user weight: 130 kg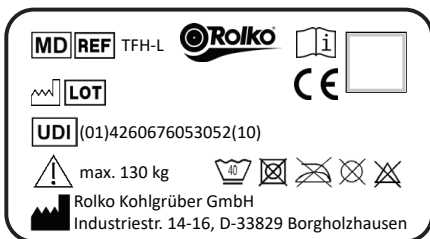

Intrinsic weight: 200 g

Washable: 40°C, max. 800 rpm

Materials used: polyester, polypropylene, polyurethane, nylon

-  Medical product
-  Item number
-  Production lot number, batch
-  Unique device identification
-  Manufacturer
-  Follow instruction manual

-  Wash at 40°C
-  Do not tumble dry
-  Do not iron
-  Do not dry clean
-  Do not bleach

As a Class 1 medical product, the product carries the CE label in accordance with Regulation (EU) 2017/745 (MDR) and thus meets all German and European safety requirements applicable to the product. The product meets the relevant sections of the DIN EN ISO 21856 standard. The EC Declaration of Conformity is available for download at [instructions.rolko.com](http://instructions.rolko.com).

**Instruction manual: Rolko-EasyTransfer leg strap - TFH-L / dated 2024-03-28 / Rev. 01**

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Basic UDI DI: 426067605TGEASYTRANSFER38

Reproduction, in whole or in part, only with prior permission of the publisher. All rights reserved. Subject to change due to technical development. All technical data is nominal data and subject to tolerances due to construction and manufacture.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen