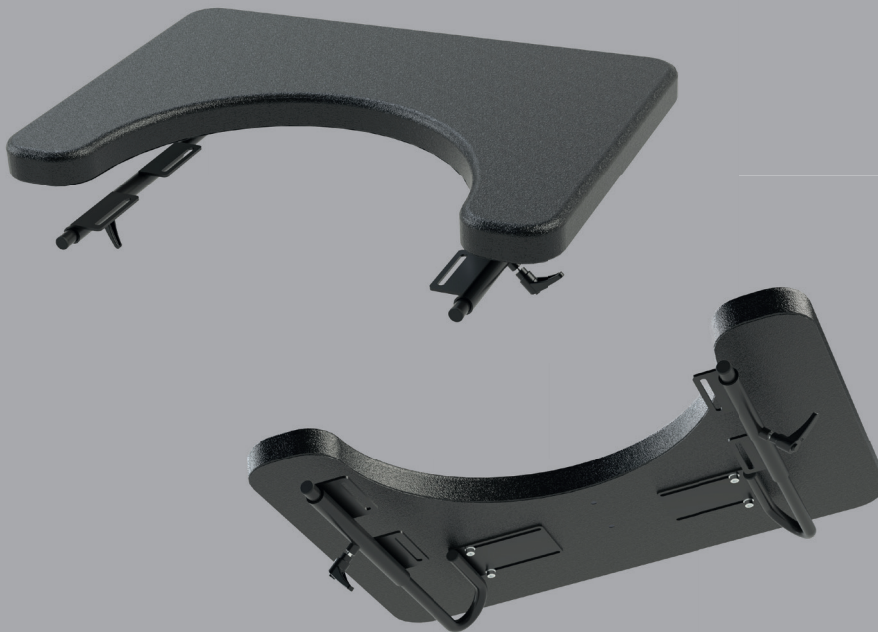
Gepolsterter Therapietisch
Padded therapie table
Gepolsterde therapietafel
Polstret terapibord
Table de thérapie rembourrée
Tavolino da terapia imbottito
TP620-POL-U

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01. Application and purpose

The padded therapy table TP620-POL-U is typically used in elderly care, hospital and rehabilitation settings. It is intended exclusively for use on wheelchairs and seat shells or seat shell underframes. It serves as an arm support and as a surface during meals, work, therapy and play.

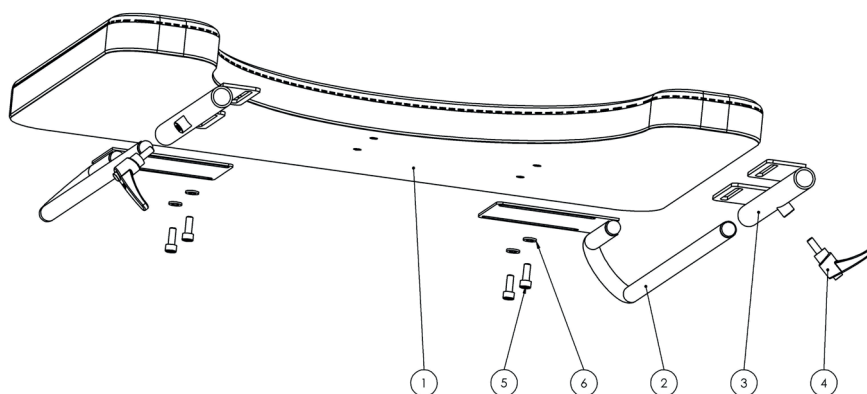
02. Contraindication

The therapy tables may only be attached to the wheelchair by qualified personnel. Care must be taken to ensure that the armrests are suitable for the type of mounting. If this is not the case, the therapy tables must not be used. The therapy tables are not suitable for restraining or securing the person in the wheelchair. They must never be mounted on the wheelchair during transport in motor vehicles.

03. Safety instructions

- These instructions for use must be read and observed before use.
- Application and assembly: The therapy table is intended only for use with armrests. It may only be assembled by qualified personnel. Before use, check that the therapy table and the mounting are firmly seated.
- Load capacity: Observe the permissible load capacity! There is a risk of breakage! See section „Technical data“.
- Driving with the table mounted: Remove all objects from the therapy table! There is a risk of injury from falling objects and a risk of burns, e.g. from hot food.
- Collision while driving: Possible collisions while driving with the therapy table mounted pose a risk of injury (pinching, crushing).
- Use in vehicles for transporting disabled persons: The therapy tables are not approved for use in vehicles for transporting disabled persons. Serious injuries may occur. Remove the therapy table and stow it safely in the vehicle.
- Breathing difficulties: Therapy tables adjusted too tightly or the disabled person tipping forward can restrict breathing. Do not leave the user unattended and inform them of the risks!
- Avoid prolonged exposure to sunlight and observe the notes in the section „Cleaning and disinfection“.
- The instructions for use are part of the therapy table and must be handed over with it if the product is passed on.
- Serious incidents in connection with this product must be reported to Rolko Kohlgrüber GmbH and to the competent authority of the EU member state in which the user is established.

04. Assembly

- The receptacles for the frame (3) are screwed under the armrests of the wheelchair.
- The frame (2) is loosely screwed to the padded board (1) using the M6 screws (5) and M6 washers (6).
- The frame (2) is adjusted to the appropriate width on the left and right using the mounted receptacles (3), and the M6 screws (5) are firmly tightened.
- The frame (2) of the fully assembled therapy table is inserted into the receptacles (3) and pushed in to the appropriate depth. The therapy table is fixed in depth with the clamping levers (4).
- To remove the therapy table, release the clamping levers (4) and pull the therapy table out of the receptacles. The receptacles can remain on the wheelchair.

The adjustable width, measured from centre to centre of the armrests, is min. 400 mm to max. 570 mm.

05. Cleaning and Disinfecting

Clean and disinfect the therapy table before first use and before each reuse. Disinfect the tabletop regularly, but at least before each change of patient. Care and cleaning: For easy cleaning, the therapy table can be wiped with a damp cloth, if necessary with the addition of a mild cleaning agent. Use lukewarm water and, if required, non-abrasive, pH-neutral, commercially available household cleaners. Please also observe the following point on disinfection/sterilisation. Only use a towel-damp cloth. Disinfection / sterilisation: We recommend the cleaning agents and disinfectants listed below for cleaning the therapy table, in consultation with their manufacturers. They may only be used in

Instruction Manual

accordance with the manufacturer's specifications! Observe the relevant product information, in particular regarding dosage and possible hazards when combined with other agents. Before using other cleaning agents or disinfectants, always consult the manufacturer of the agent. Due to the large number of cleaning agents and disinfectants on the market, as well as constant innovations, we cannot provide a comprehensive guarantee of compatibility. If in doubt, please contact the distributor or manufacturer, as the use of harmful substances can cause premature wear or material failure and thus create hazards.

The following disinfectants are permitted: From the biguanide group: Incidin Plus. From the peroxide compound group: Apesin AP100, Dismozon plus, Dismozon pur, Incidin active, Perform, Terralin paa, Wofasteril. From the ammonium chloride group: Mikrozid sensitive liquid, cosiMed disinfectant.

06. Maintenance

Worn or damaged parts must be replaced immediately. Pay particular attention to wear of the tabletop cover!

07. Re-use

The product is suitable for reuse. Environmental influences, UV radiation, mechanical stress, natural ageing or careless handling can cause premature wear over which the manufacturer has no direct influence. A thorough basic cleaning is required for reuse, see section „Cleaning and disinfection“.

08. Disposal

Disassemble the therapy table as far as possible. Small and plastic parts can be disposed of with household waste. Larger metal parts are accepted by recycling facilities.

09. Technical data

Item no.:	TP620-POL-U
Tabletop dimensions:	620x450x37 mm
Net weight:	3,4 kg
Material:	PC, PA6, PVC, PU, POM, PE, galvanised steel / painted black
Load capacity:	Max. 10 kg

Label

Manufacturer

Follow the instructions

Production date

Item no.

Batch

Medical device

Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 20417. The EU declaration of conformity is available for download at rolko.com.

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen