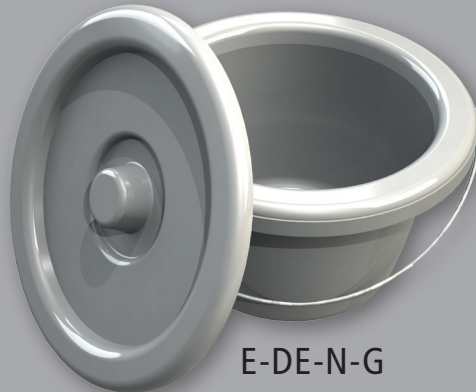
E-DE-G

E-DE-N-G

## Toiletteneimer Toilet Buckets

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### Contents

01. Purpose and Indication .....	3
02. Technical Specifications .....	3
03. Safety and Application Instructions .....	3
04. Cleaning and Disinfecting .....	3
05. Maintenance .....	3

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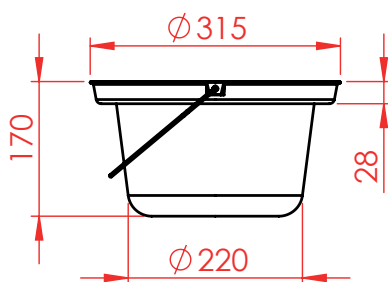
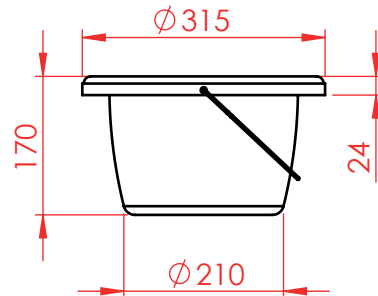
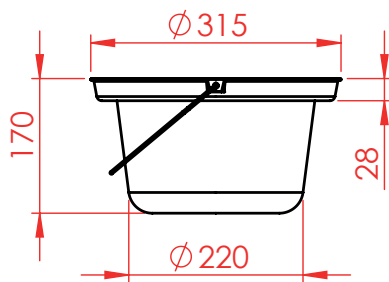
## 01. Purpose and Indication

Toilet buckets are hung on the holder under the seat opening of toilet chairs. These can be used similarly to normal toilets. Excreta are collected in the toilet bucket and later disposed of in the toilet. Toilet buckets are mainly used in the nursing, hospital and rehabilitation area, when patients are not able to use the regular toilet anymore due to physical restrictions.


## 02. Technical Specifications

Article No.: E-DE-G (with water gutter)  
E-DE-N-G (without water gutter)  
Colour: light gray  
Material: plastic (polypropylene)

Measurements in mm:

E-DE-G

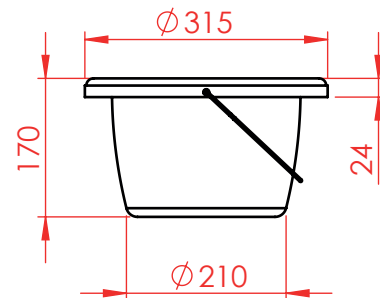E-DE-N-G

## 03. Safety and Application Instructions

- These operating instructions must be read before use.
- While closing the bucket, slightly press the lid onto the bucket so that the lid attaches to the bucket and closes it airtight and therefore odourless. Alternatively, fill the gutter with water in order to lock the odour up (only applies to the model E-DE-G).
- Pay attention to the proper holder of the toilet chair.

## 04. Cleaning and Disinfecting

- Please clean the product for max. 5 minutes at a maximum temperature of 85 °C (185 °F).
- The use of boiling water and steam is not allowed.
- During the cleaning, there must not be any pressure on the bucket as there might occur distortions otherwise.
- The use of household disinfectants is allowed.

## 05. Maintenance

- Check the bucket for damages like cracks or holes regularly.
- Follow the cleaning and disinfection instructions.

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**Operating Instructions: Toilet Buckets / Revision: 2016-02-18**

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved.

Alterations for reasons of technical development reserved. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances. Errors excepted.

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**© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen.**
**© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Germany**